LANAFORM

WELLNESS

Heat & Fold Spa

FOLDABLE HEATING
FOOT SPA

FOLDS
FLAT

HEATING
FUNCTION

ACUPRESSURE
MASSAGE SURFACE

REFLEXOLOGY
MASSAGE

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ENGLISH

INTRODUCTION

Thank you for choosing the "Heat & Fold Spa" by LANAFORM. With its relaxing, stimulating and reinvigorating effect, the "Heat & Fold Spa" will help make stress, fatigue and muscle pain disappear. Fitted with detachable massage rollers and acupressure points, the "Heat & Foot Spa" will ensure you have an excellent massage and will stimulate your reflex points. It also comes with pedicure accessories so you can give your feet a real treat before or after your massage. Plus, it will help your circulation with its heating function and infrared lamp. Finally, owing to its flexible design, you will be able to store it easily.

▲ PLEASE READ ALL THE INSTRUCTIONS BEFORE USING YOUR "HEAT & FOLD SPA", ESPECIALLY THIS BASIC SAFETY INFORMATION:

▲ This device contains a heating element. Be aware of the risks of burning if you are sensitive to heat.

▲ If any liquid escapes, stop using the device immediately.

- Always follow these instructions when using this appliance.
- Check that your mains voltage is the same as that of your appliance.
- This appliance can be used by children aged from 8 years and above and per-

1

2

3

sons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

- Do not use accessories that are not recommended by LANAFORM or that are not supplied with this equipment.
- If the power cable is damaged, it must be replaced by a special or similar cable available from the supplier or its after-sales department.
- Do not use this device if the plug is damaged, if it is not working properly, if it has been dropped on the floor, or if it is damaged or has fallen into water. In this event, have the device examined and repaired by the sup-

plier or its after-sales department.

- Do not carry this appliance by its power cable or use the cable as a handle.
- Always unplug the appliance after use, and allow it to cool down before moving or cleaning it.
- If it falls into water, unplug it immediately before retrieving it.
- Electrical appliances should never be left unattended when connected. Unplug it when you are not using it.
- Keep the power cable away from hot surfaces.
- Never use this appliance in a room in which aerosols (sprays) are used or in a room where oxygen is administered.
- This unit is intended only for family use.
- This equipment is suitable for indoor use only, on a flat surface).
- Never block the air vents, and never place the foot spa on an unstable surface such as a carpet or mattress. This could prevent

ventilation. Make sure that the ventilation holes are not clogged with hair, fluff, dust, etc. and do not insert any objects into them.

- Check the water temperature before use.
- Do not fill the "Heat & Fold Spa" above the "MAX" level shown on the flexible part of the reservoir on the left.
- Do not use this equipment near a shower, pool or any other location where water is present.
- Never attempt to connect or disconnect this appliance if you are in contact with water or in a damp atmosphere. This could cause an electric shock.
- Do not immerse the equipment in water or any other liquid in order to clean it.
- Use the "Heat & Fold Spa" only when seated.
- Never use this device if you are drowsy or while asleep.
- After each use, be careful not to fall if

you are on a smooth surface.

- If water is leaking from the appliance, stop using this product immediately and return it to your supplier or its after-sales department.
- If you are concerned about your health, consult a doctor before using this equipment.
- If you experience any pain when using this device, stop using it immediately and consult your doctor.
- Never use this appliance if your feet are swollen or inflamed or if you have a skin rash.
- This product is a non-professional massage device designed to relax tired muscles. Do not use this equipment as a replacement for medical treatment.
- This device is designed to be used for a short period only. We recommend that you use it for a maximum period of 15 minutes. It must then be switched off so that the motor can cool

down for at least 15 minutes.

- Use of the "Heat & Fold Spa" device without consulting your doctor is strongly discouraged for pregnant women, those subject to thrombosis or suffering physical pain, the cause of which has not been defined, as well as for those wearing a pacemaker. Use of the device is also strongly discouraged for diabetics.

FEATURES

- Foot spa
- Heating function up to 41°C
- Infrared lamp
- 2 detachable massage rollers
- 3 interchangeable pedicure accessories
- Stimulating acupressure surface area
- Fits up to size 44
- Can be folded up for easy storage

DESCRIPTION ■ 1

- 1 On/off button
- 2 Reservoir
- 3 Water jet strips
- 4 Acupressure points
- 5 Detachable massage rollers ▲ [L/R]
- 6 Interchangeable pedicure accessories
 - a Massage accessory
 - b Brush
 - c Abrasive head
- 7 Heat resistor
- 8 Infrared lights
- 9 Maximum water level
- 10 Splash guard

- 11 Power cable
- 12 Power supply
- 13 Rubber feet
- 14 Metal handles

INSTRUCTIONS FOR USE

Assembly ■ 2

- 1 To unfold your spa, pull on the 2 metal handles located in the base and pull them out completely. (1)
- 2 Put one hand in the reservoir and pull firmly on the side of the spa including the splash guard to lift it up. (2)
- 3 Lift up the metal handle on the same side to jam its end under the rim of the spa. (2)
- 4 Then pull on the opposite side to unfold the spa completely. (3)
- 5 Lift up the metal handle on the same side to jam its end under the rim of the spa. (3)
- 6 The "Heat & Fold Spa" is ready for use.

Use ■ 1

- 1 Put the appliance on a flat, stable, hard surface with a non-slip mat alongside it to avoid any risk of falling.
- 2 Put the accessories you want to use in the reservoir (see below).
- 3 Fill the tank (2) with water at the required temperature up to "MAX" on the flexible section of the reservoir to the left (9). Never fill above this level.

▲ Always immerse the heating element (7).

- 4 Plug the device into the mains. Make sure too that your hands are not wet and your feet are not in the water.
- 5 Sit on a chair and put your feet in the water. Never stand up in the "Heat & Fold Spa".
- 6 Then press the on/off button to start the spa (1).

▲ Never turn on the device without water in the tank!

- 7 Press a second time to start the heating function and infrared lamp. The two infrared LEDs will light up (8).
- 8 When the heating function has been activated, the water will gradually heat up. The lower the starting temperature of the water, the longer it will take to heat up again. Fill the reservoir with tepid water if you want to speed up the reheating process.

- 8 The massage does not generally take more than 10 to 15 minutes.
- 9 Once the massage has finished, press the on/off button again to turn the device off.
- 10 Finally, empty the contents of the reservoir from one of the 2 opposite ends to the splash guard lid. (Fig. 3)

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Accessories ■ 1

To have a reinvigorating, stimulating massage, use the acupressure points (4) and the 2 detachable rollers (5), putting light pressure on those points. You also have a pedicure set consisting of:

- A massage accessory providing a pleasant massage for the feet and encouraging blood flow (6a).
- A brush stimulating the reflex areas of the soles of the feet (6b).
- A callus remover is used to remove dead skin cells as well as corns (6c).

Install the accessory you want in the mechanism provided for this purpose (6).

MAINTENANCE

Storage

- Remove the detachable accessories from the reservoir.
- Fold up your spa:

- 1 Unclip the 2 metal handles from under the rims.
- 2 Press firmly on the rims to flatten the device.
- 3 Push the metal handles back into the base.

- Put the massage device in its box or in a dry place.
- Do not put the device in contact with sharp or pointed items that might damage the outer surface.
- Do not wrap the power cable round the device. (This may cause wear and tear and break the cable).

Cleaning

- Unplug the device and allow it to cool before cleaning.
- Do not immerse the device in water or any other liquid to clean it. Use a gentle detergent and a soft, slightly damp sponge. Pat dry with a dry, clean cloth.
- Never use abrasive detergents, brushes or solvents to clean it.

TECHNICAL FEATURES

- **Voltage:** 220-240V
- **Frequency:** 50Hz
- **Power:** 500W

ADVICE REGARDING THE DISPOSAL OF WASTE

All the packaging is composed of materials that pose no hazard for the environment and which can be disposed of at your local sorting centre to be used as secondary raw materials. The cardboard may be disposed of in a paper recycling bin. The packaging film must be taken to your local sorting and recycling centre.

When you have finished using the device, please dispose of it in an environmentally friendly way and in accordance with the law.

LIMITED WARRANTY

LANAFORM guarantees this product against any material or manufacturing defect for a period of two years from the date of purchase, except in the circumstances described below.

The LANAFORM guarantee does not cover damage caused as a result of normal wear to this product. In addition, the guarantee covering this LANAFORM product does not cover damage caused by abusive or inappropriate or incorrect use, accidents, the use of unauthorized accessories, changes made to the product or any other circumstance, of whatever sort, that is outside LANAFORM's control.

LANAFORM may not be held liable for any type of circumstantial, indirect or specific damage.

All implicit guarantees relating to the suitability of the product are limited to a period of two years from the initial date of purchase as long as a copy of the proof of purchase can be supplied.

On receipt, LANAFORM will repair or replace your appliance at its discretion and will return it to you. The guarantee is only valid through LANAFORM's Service Centre. Any attempt to maintain this product by a person other than LANAFORM's Service Centre will render this guarantee void.

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